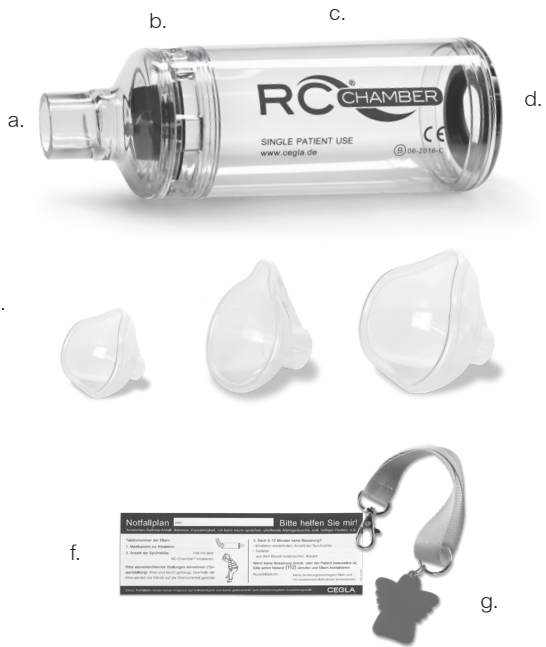
RC-CHAMBER®

Optimiert die Inhalation mit Dosieraerosolen
 Optimized inhalation with metered dose inhalers
 Optimiza la inhalación con inhaladores de cartucho presurizado
 Inalazione ottimizzata con inalatori-dosatori
 Inhalation optimisée avec les inhalateurs-doseurs

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INSTRUCTIONS FOR USE – ENGLISH

THANK YOU FOR CHOOSING THE RC-CHAMBER® THERAPY.
NOTE

Please read the Instructions for Use carefully before using RC-Chamber® for the first time. Please also note the instructions for the metered-dose inhaler. Please keep these instructions for future reference. Please note that the device is designed for single-patient use only! All RC-Chamber® parts can be sterilized in boiling water and disinfected in a steam sterilizer as well as cleaned in the RC-Clean® cleaning bag. (To prevent the build-up of lime-scale, we recommend boiling the device in distilled water.) Store RC-Chamber® in a dry, dust-free environment and protect it from sunlight. We recommend the exchange of the RC-Chamber® after one year of using. In Germany the RC-Chamber® is reimbursable once a year on submission of a medical prescription. Disposal: RC-Chamber® and its accessories may be discarded with household waste unless otherwise stipulated by the regulations of the respective country of use.

! WARNINGS

Do not allow children to use with RC-Chamber® without supervision. An adult should always be present when therapy is administered to children and those requiring assistance. The tool contains small parts. Please keep it well away from babies and small children in order to avoid the risk of choking. Please clean the device prior to using it for the first time. Clean the device once per week if used regularly, or more frequently as needed. If the device is visibly soiled, clean RC-Chamber® immediately prior to or after use. Replace RC-Chamber® after one year if used daily. Please ensure that RC-Chamber® is used properly according to the Instructions for Use. Incorrect application (off-label use) results in exclusion of liability for the manufacturer.

CONTENTS

The package contains RC-Chamber®, consisting of:
 a. Mouthpiece
 b. Valve & inhalation controls - Cross Valve Technology™
 c. Spray entrance chamber
 d. Parte posterior: clear plastic ring with integrated rubber adapter
 e. A mask is also included with the following articles:
 • RC-Chamber® with mask for infants
 • RC-Chamber® with mask for young children
 • RC-Chamber® with mask for adults and older children
 f. Emergency plan (to stick to the back of the security bag)
 g. Guardian angel pendant (to attach to the security bag, perforation above the pressure lock necessary)

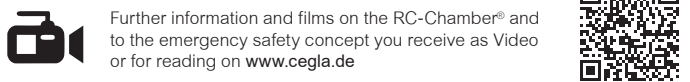
Materials:

Chamber: Makrolon polycarbonate resin
 Back piece: Santoprene
 Valve & inhalation controls as well as masks: Silicone
 All materials are latex-free.

Safety breathing concept

A reliable companion throughout the day and in an emergency

Included in delivery RC-Chamber® including emergency plan and guardian angel pendant. Drugs as well as the Peak Flow Meter Personal Best® must be ordered separately.

AVAILABLE PRODUCTS

- RC-Chamber® with mask for infants up to 1 year
Article no. 30100 | PPN 111 171 099 278
- RC-Chamber® with mask for young children, 1-5 year
Article no. 30200 | PPN 111 171 100 087
- RC-Chamber® with mouthpiece for adults and older children
Article no. 30300 | PPN 111 171 102 343
- RC-Chamber® with mask for adults and older children
Article no. 30400 | PPN 111 171 101 777
- RC-Chamber® for tracheostomized patients
Article no. 30450 | PPN 111 273 164 894

MASKS

Masks available separately.

- Infant mask
Article no. 30110 | PPN 111 416 498 106
- Child mask
Article no. 30210 | PPN 111 416 499 893
- Mask for adults and older children
Article no. 30410 | PPN 111 416 500 605
- RC-Tracheal mask for tracheotomized patients
Article no. 3012

ACCESSORIES AND ADDITIONAL PRODUCTS

RC-Clean®
 Reusable microwavable cleaning bag. Easy and quick cleaning of medical products as well as baby items.*
 Use only for microwave safe products.
 Article no. 5000 | PPN 111 075 103 265

* Report on microbiological testing available on request.

Replacement parts and additional products are available in pharmacies and medical supply stores.

To constantly improve our level of service, we welcome your feedback. You will find helpful tips for living with respiratory diseases on our platform www.leichter-atmen.de. We look forward to your visit.

All trademarks and registered trademarks belong to the company CEGLA Medizintechnik GmbH, Germany. In some instances, product graphics may vary from actual product models, sizes and/or colors listed below.

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PURPOSE

RC-Chamber® is an inhalation device developed specifically for medication spray therapy, so-called MDIs, metered-dose inhalers, and is used in the treatment of respiratory diseases. The flexible rubber adapter is compatible with all conventional dose inhalers.

When using RC-Chamber®

- it is no longer necessary to coordinate the release of the aerosol spray and inhalation of the drug
- large, non-respirable particles are filtered out. This helps prevent irritation that may lead to yeast infections in the mouth and throat.

- A** Without RC-Chamber® most of the drug is deposited in the mouth and throat.
B With RC-Chamber® delivery of the drug to the lungs improves.

INSTRUCTIONS Use

Please inspect the device for foreign bodies and damage prior to use. If any parts are missing or damaged, please replace them immediately.

Attach the mask securely until it locks in.

Remove the sealing cap from the MDI mouthpiece and shake the aerosol spray container vigorously prior to each use.

Place the MDI mouthpiece as shown in the illustration into the rubber adapter provided on RC-Chamber®.

Make sure the connection to the rubber adapter is straight and tight.

RC-Chamber® with mouthpiece or mask

Rodee your lips tightly and securely around the mouthpiece or place the mask tightly over nose and mouth.

Tip

To familiarize children with the use of RC-Chamber®, it may help to release the aerosol spray into the chamber before putting the mask on the child.

RC-Chamber® for tracheostomized patients

Connect RC-Chamber® directly to the cannula or to the RC-Mask Tracheostoma in the event of an open tracheostoma.

RC-Chamber® with mouthpiece

- I. Push the cartridge of the MDI down in order to release the spray into RC-Chamber®.
- II. Inhale and hold your breath for 5 to 10 seconds. This optimizes the therapeutic effect of the spray.

RC-Chamber® with mask

- I. Push the cartridge of the MDI down in order to release the spray into RC-Chamber®.
- II. Hold the RC-Chamber® mask combination to the face for 4 to 6 breaths. This procedure can even be administered to a sleeping child. The number of breaths can be seen in the valve movement.

RC-Chamber® for tracheostomized patients

- I. Push the cartridge of the MDI down in order to release the spray into RC-Chamber®.
- II. Inhale and hold your breath for 5 to 10 seconds. This optimizes the therapeutic effect of the spray.

If the treating physician prescribed several spray applications, always only release one puff into the chamber for every inhalation procedure. Please wait approximately one minute before repeating steps IV to VI!

INSTRUCTIONS Cleaning

After using a mask, detach it from RC-Chamber®. Remove the clear plastic ring with the attached integrated rubber adapter by pressing your thumb deep into the opening. Use force to pull off the back piece.

Cleaning and disinfection

1. **Wash** the parts in a solution of lukewarm clean water and standard liquid soap detergent. The RC-Chamber® is dishwasher safe. Due to the possibility of leftovers we do not recommend this cleaning method.
2. **To disinfect** all parts, place them in boiling water for approx. 5 minutes. Please use a clean pot with sufficient fresh drinking water.

The RC-Clean® cleaning bag may also be used to clean the device (see accessories).

Shake to remove excess water and allow all parts to air-dry completely before reassembly. **Do not wipe dry!**

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Reassemble when all parts are completely dry. Attach the mask securely until it locks in. Make sure that the rubber adapter is attached in the same direction as the mouthpiece. Store RC-Chamber® with or without mask in a dry, dust-free environment, for example in the original RC-Chamber® package. Protect from sunlight.

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